


MANULET

HELP FOR ACTIVE BEDRIDDEN PEOPLE



Manual transfer

- with focus on professionalism

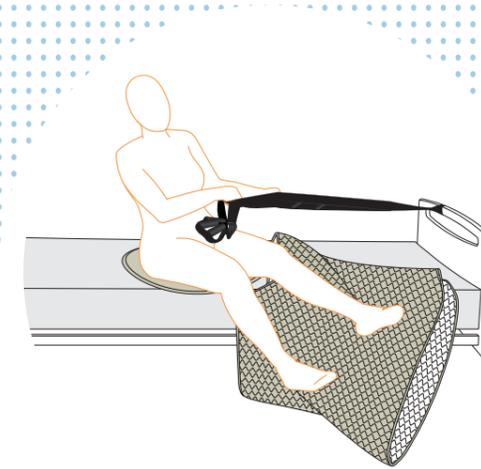

UTILIZING EXISTING FUNCTIONS

When a person's strength is compromised due to illness it becomes difficult to move in bed. Often a helping hand is needed when it comes to bed mobility. It can be a huge challenge for such persons to turn on their sides, to transfer from a lying to sitting position and to get their legs up onto the bed.

But where do they find a helping hand? Should it only be a family member or carer offering assistance using their own physical strength.

It is a known fact that the care sector has a number of Occupational Health challenges. The challenges are in particular related to manual moving and handling of bedridden persons. Relatives and nursing staff expose themselves to risks and unnecessary strains when they use their physical strength to compensate for the lack of function of bedridden persons.

We at Vendlet believe that it is a question of supporting and utilizing whatever strength the person still has. It is therefore a question of offering support to those who are still capable of self help.

“It is a question of supporting and utilizing the strength the persons still has.”

PRESERVING DIGNITY

Preserving ones dignity is also a reason why many people prefer to manage on their own when it comes to personal things like dressing and undressing. We believe in helping people preserving their dignity by offering them options for managing on their own for as long as possible.

Basically, it is a question for people controlling their own lives; feeling independent and not dependent on help from others. Many people experience it as a major intervention in their lives and self-image when they suddenly need help from the care sector. It is also a major intervention in family life.

Therefore, we help people preserve their dignity when we offer them the option of managing on their own for as long as possible.

THE PHILOSOPHY BEHIND MANULET

Peter Maindal the physiotherapist who has developed the MANULET series, states we should all endeavour to avoid manual moving and handling. Transfers where the nursing staff lift, slide or push have a negative impact on both the carer and patient resulting in injury and pain for both parties.

For this reason products are required that are simple to use and easy to understand.

The MANULET series consists of four different products that can be used individually or in conjunction with each other. They all have the purpose of supporting the bedridden person according to their level of self help . The combination of friction reducing, good instructions of use and technique will make up for the lack of function of the beridden person.



MANULET Easy Tube

MANULET Easy Tube is used to reduce friction under the bedridden person.

MANULET Easy Tube has a super smooth inside and skin-friendly, soft cotton jersey on the outside. The two layers are quilted together to achieve stability of form and optimum function.

MANULET Easy Tube provides great flexibility in use. It is used for many different transfers i.e a transfer higher up in bed, for turning around in bed and for a person getting their legs up onto the bed.

The unique design of MANULET Easy Tube means it keeps its own shape and does not bunch up under the user and therefore helps preserve tissue integrity.

MANULET Easy Tube can also be used in combination with other products i.e VENDLET, MANULET Multi Grip or MANULET Turn Table.

ITEM NO.: 5000101
MAX. LOAD: 300 kg
SIZE: L 110 x W 70 cm


Materials

FABRIC: Cotton and nylon
FILLING MATERIAL: Fibre filling

Washing & cleaning



Do not add fabric conditioner

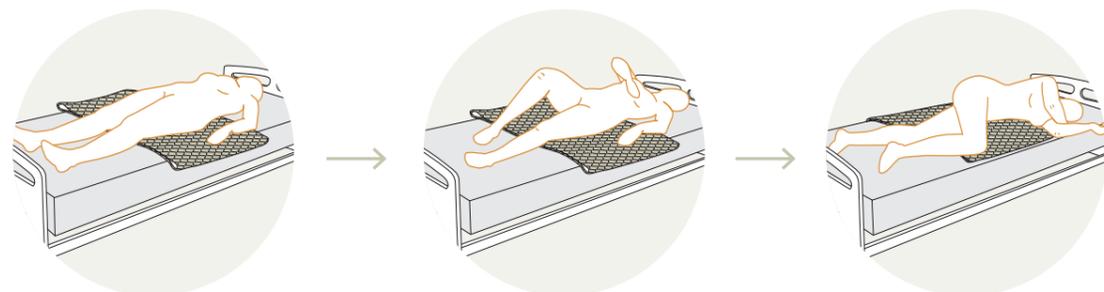

MANULET Turn Table 45 and 60

MANULET Turn Table is used to make it easier to turn around in a sitting position.

MANULET Turn Table consists of two round discs with a super smooth inside and skin-friendly padded cotton jersey on the outside. The two discs are attached to a swivel with velcro. The swivel is specially designed for very low resistance even when exposed to high loads. The swivel can easily be removed for washing.

MANULET Turn Table is placed under the pelvis of the person in a sitting position. The friction between the person and the base is eliminated and the person can turn easily. This may be of great importance e.g. when moving into and out of bed or into the car.

MANULET Turn Table can also be used in combination with other products such as VENDLET, MANULET Easy Tube or MANULET Multi Grip.

MANULET Turn Table is available in two sizes.

ITEM NO.: 5000104 / 5000105
MAX. LOAD: 8200 kg
SIZE: Ø 45 / Ø 60 cm


Materials


FABRIC: Cotton and nylon
FILLING MATERIAL: Neoprene

Washing & cleaning



Remove the swivel before washing
 Do not add fabric conditioner

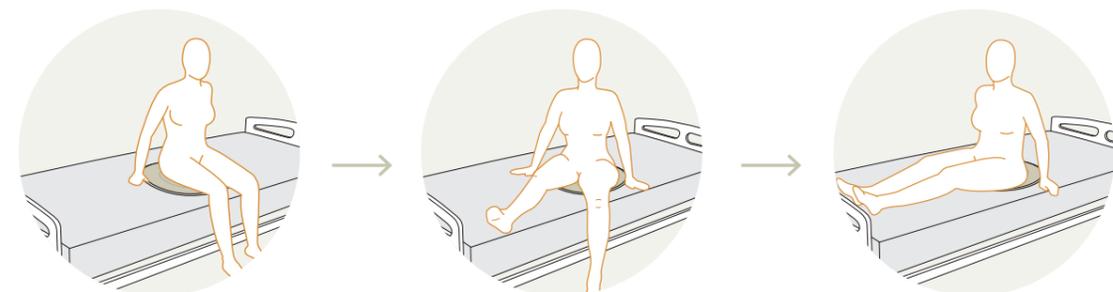

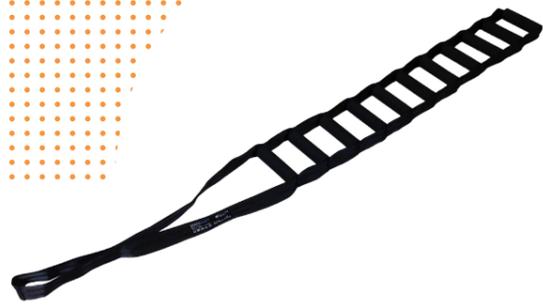

MANULET Multi Grip

MANULET Multi Grip can be used to get from a lying to a sitting position and vice versa.

MANULET Multi Grip is mounted at the foot of the bed and is placed on the mattress alongside the bedridden person.

The sturdy soft foam handles have a diameter of 3 cm, which provides an anatomically correct grip. The unique structure of MANULET Multi Grip makes the handles very stable when they are used. This results in high comfort and is of special importance to people with weak hands and wrists.

Multi Grip is very flexible as it has 11 handles and a total length of 170 cm. It offers a good grip in many different positions, which makes it easy to “climb up or down” using the Multi Grip.

MANULET Multi Grip can also be used in combination with other products such as VENDLET, MANULET Easy Tube or MANULET Turn Table.

ITEM NO.: 5000107
MAX. LOAD: 400 kg
SIZE: L 170 cm


Materials

STRAP: Polyester
HANDLES: Polypropylene tubes covered with PE foam

Washing & cleaning



Do not add fabric conditioner

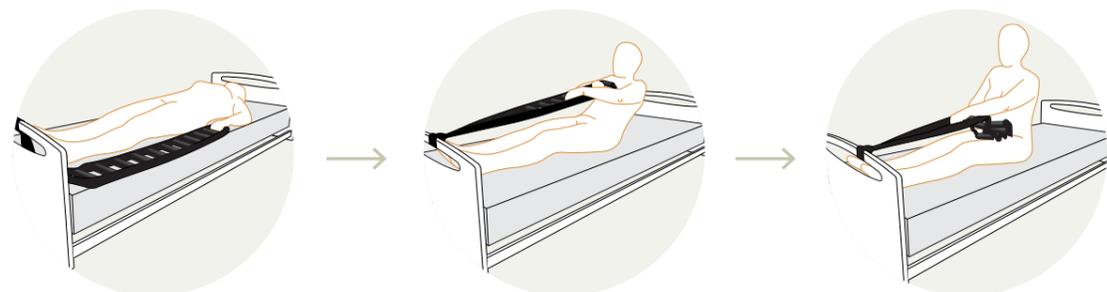

MANULET Mono Grip

MANULET Mono Grip is for people who find it difficult to turn in bed.

MANULET Mono Grip is mounted on the frame of the bed and placed on the mattress next to the bedridden person.

The sturdy and soft foam handle has a diameter of 3 cm, which provides an anatomically correct grip. It is ideal for arthritic fingers and hands. This results in high comfort and the possibility of a good, stable grip.

MANULET Mono Grip can be adjusted in length and can easily be adapted to the bedridden person and their specific situation. This means that bedridden persons can always have Mono Grip within reach and thus have the possibility of turning over by themselves onto their side.

MANULET Mono Grip can also be used in combination with other products such as VENDLET, MANULET Easy Tube or MANULET Turn Table.

ITEM NO.: 5000106
MAX. LOAD: 400 kg
SIZE: L 35 - 95 cm


Materials

STRAP: Polyester
HANDLES: Polypropylene tubes covered with PE foam

Washing & cleaning



Do not add fabric conditioner

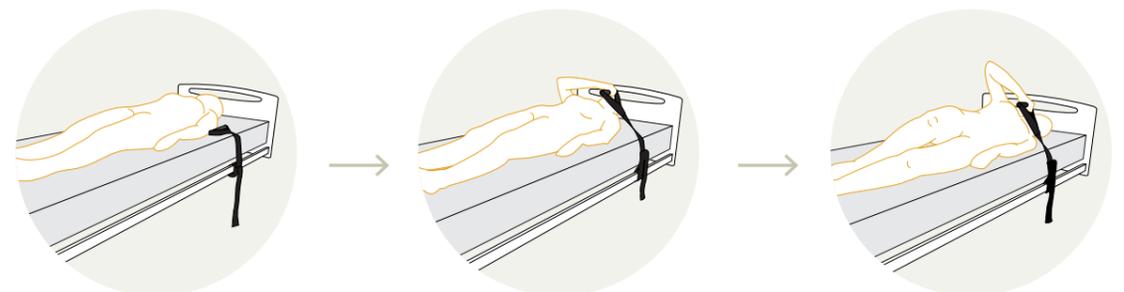

Manufacturer:

Vendlet ApS
Egelund 33
DK-6200 Aabenraa
Tel.: +45 9631 0050
vendlet@vendlet.dk
www.vendlet.com

 All rights reserved.
Copyright 2016 Vendlet ApS


VENDLƏT
VALUE IN CARING

Vers. 1.0