




R82[®]

x:panda shape

Cleaning guide

Cleaning guide

To ensure that the x:panda shape seating system stays safe for the user and their caregivers it is important to keep it clean and well functioning. Our goal with this guide is to help parents and caregivers manage everyday cleaning and maintenance of the x:panda shape seating system. This guide does not replace the user manual.*

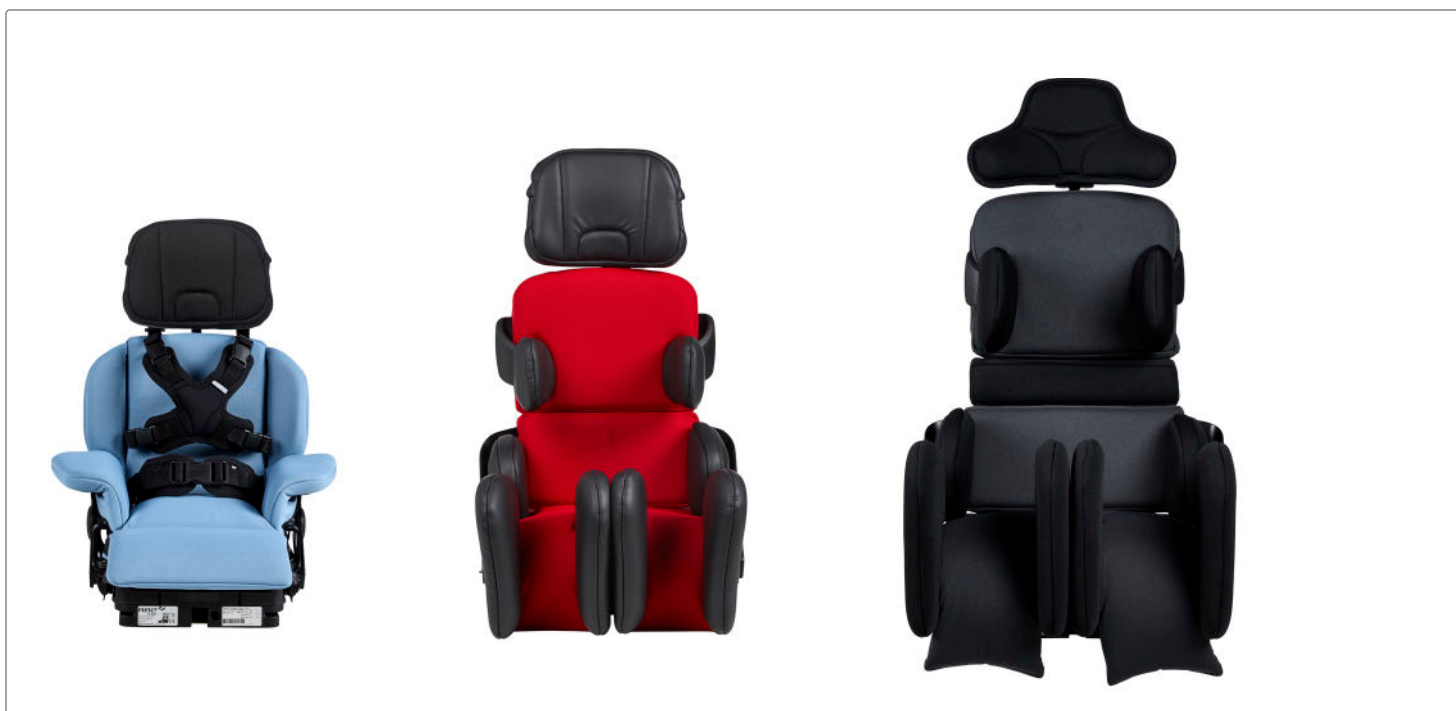

Cleaning materials

In this guide we will recommend using the following materials:

- Cloth
- Warm water
- Mild detergent/soap without chlorine
- 70% disinfectant Isopropyl alcohol solution
- Washing machine
- Laundry bag
- 5 mm Allen key



Cleaning tip!

Cushion covers, accessory covers, vest and belt can be washed at 60 °C using a mild detergent in a washing machine.

Use a laundry bag for vest and belt for protection of the plastic buckle.

During the cleaning it can be necessary to demount some of the accessories to be able to clean the parts and to clean underneath them. Consider taking photos of the settings of the product before adjusting and dismounting. You can also note down supplement to the visual markings in our sheet with important markings:



Click or scan to download the sheet with important markings from our website.



* This guide does not replace the user manual

Cushion covers, accessory covers, vests and belts

If the cushion cover, accessory cover, vest and/or belt need to be cleaned, then take photos or note down the setting of the fix locks used for mounting of vest and belt.

Before washing:

- Remove vest and belt
- Remove the cushions (seat, lower back, upper back) and remove the back plate from lower and upper back cushion. Separate the cover from the foam
- Remove covers from accessories (e.g., inner/outer knee supports, pommel, hip/pelvis supports, side supports and head support)

When cleaning underneath the seat cushion it can be necessary to loosen or remove some of the accessories. Not all seats will have the same accessories and there can be parts that can be skipped. It is not recommended to wash the foam part.



Cover and foam for upper back cushion and seat cushion.



Fix locks at the lower back and upper back for mounting of vest.



Fix locks at the seat base for mounting of belt.

Wedges

If wedges are mounted at the seat, then do the following:

- Note in which position they are mounted. Right and left might be different.
- Consider if it is enough to loosen them or if they must be demounted
- If demounted, is there a mounting kit for wedges underneath?
- Clean the wedges



Position of the wedge and mounting kit for wedges.



Loosen (1) or demount (2) the wedge.



Cleaning tip!

The wedges and the inner and outer knee supports can be cleaned using a cloth, warm water, and a mild detergent/soap without chlorine. Let all parts dry before use.

The wedges can also be disinfected with a 70% disinfectant Isopropyl alcohol solution. It is recommended to clean with warm water and soap and let it dry before disinfection.

Inner and outer knee supports

If inner and outer knee supports are mounted at the seat, then do the following:

- Consider if it is necessary to demount to clean
- If it is necessary to demount, note in which position they are mounted
- Demount and clean the inner and outer knee supports



Position of the inner and outer knee supports.

Advanced seat plates

It can be necessary to clean underneath the seat plates as well, then do the following:

- Note in which position they are mounted. Are there a difference between left and right
- Consider if it is enough to loosen them or if it is necessary to demount them
- Clean the seat plates



Cleaning tip!

The seat plates and other parts of seat can be cleaned using a cloth, warm water, and a mild detergent/soap without chlorine. Let all parts dry before use.

The seat plates and other parts of seat can also be disinfected with a 70% disinfectant IPA solution. It is recommended to clean with warm water and soap and let it dry before disinfection.



Angle and width position of the seat plates



Depth position of the seat plates

Mounting

When all parts have been cleaned, they must be mounted correctly again. Mount and adjust in the following order:

- Seat plates
- Inner and outer knee supports
- Wedges
- Covers for accessories
- Cushions
- Vest and belt



Explore our how-to-videos to learn more about the adjustments on the x:panda shape seating system and how to do the cleaning.

For the latest news and continuously updated product information – visit: www.etac.com

R82 A/S

+45 79 68 58 88

R82@etac.com

www.etac.com