

# VENDLET Basic

## QUICK GUIDE

### HOW TO USE VENDLET BASIC



Raise the bar




Lower the bar



Reposition the client

1. Turn the VENDLET on by double clicking the  button on the handcontrol. You will hear a beep and see a green light come on over the power symbol. When the hand control hasn't been used for 90 seconds, the system will turn off.
2. Raise the bar: Lift the bar with both hands until it locks at both corner posts. Remember to slacken the turning sheet by pressing the  or  button before raising the bar.
3. Lower the bar: Lift the bar and press the grey button on the corner post at the head end. When it is released you can lower the head end of the bar. Do the same at the foot end.
4. The client is repositioned using the hand control. NB: The bars on the VENDLET Basic turn together. If the red bar pulls, the blue bar starts slackening, and vice versa.

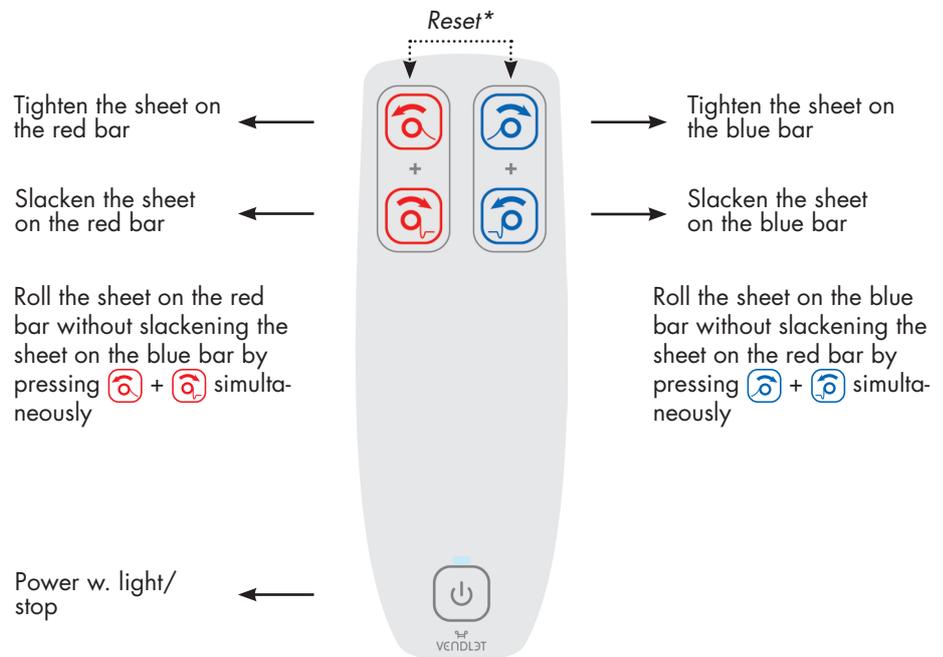

5. If the client must use side rails, slacken the sheet and raise the bars.
6. If the client should not use side rails, lower the bars.



The  button works as a stop button

Always focus on the client when using VENDLET.



\* If there is an acoustic alarm, reset the system by simultaneously pressing the two top buttons on the hand control. You will hear 10 rhythmic beeps. Press and hold the buttons until the beeping stops.



**VENDLET must be used and installed in accordance with the manual**



### ATTACHING THE SLIDE SHEET



1. Fold the corner elastic straps around the mattress.
2. Check that the surface is smooth and even without creases and folds. Make sure that slippery side is facing upwards.
3. Fasten the velcro straps around the edge of the mattress support platform. It is important that the straps are fastened around the edge of the platform under the mattress and not around the bed frame as this will prevent the back, thigh, and foot parts of the bed from being raised.



### ATTACHING THE TURNING SHEET W. QUICK TAG



1. Place the turning sheet in the middle of the bed.
2. Press the edge strip of the turning sheet into the groove on one of the bars; begin at the head end of the bed and work your way down to the foot end.
3. The turning sheet must always be placed OVER the bars to prevent fingers, hair ect. of the patient from getting caught under the sheet. If the sheet is placed under the bars, the pulling power will be reduced and the motors will switch off when the turning sheet tightens.
4. Press the  or  button on the hand control and allow approx. two-thirds of the sheet to be pulled in by the bar.
5. Press the edge strip at the opposite edge of the turning sheet into the groove on the other bar.
6. Use this bar to pull in the turning sheet by pressing the  or , button until the sheet covers the mattress evenly with the red line at the centre of the mattress.
7. Should the turning sheet be too loose on one side, you can tighten the sheet by pressing  +  simultaneously or  +  simultaneously.



*VENDLET must be used  
and installed in accordance  
with the manual*

Scan the QR code to watch  
videos on how to attach  
the sheets

