



etac[®]
Creating Possibilities

R82 

R82 Swan Curo

Ensuring great comfort and easy caregiving!

R82 Swan Curo

The Swan Curo is a mobile toilet and bath chair with a versatile functionality, ensuring everyday convenience for user and caregiver. It provides children and youngsters with special needs a comfortable, relaxed and hygienic toileting experience.

Self-care and dignity

The Swan Curo toilet and bath chair has a minimalist design and a small footprint, making it suitable for various bathrooms and homes. The shell seat is carefully shaped for high comfort and has smooth surfaces, making it easy for the caregiver to clean. The versatile function can minimise the need for transfers, which often include using a hoist and sling.

Involvement in daily toileting activities and learning the basic skills of personal hygiene can have a positive effect on self-care and promote dignity - thereby promoting quality of life for the child.



Features

The Swan Curo is available in 4 different sizes. Size 1 is available with and without castors and size 2 - 4 comes with castors. The toilet and bath chair is suitable for users from 2 years and up. Accessories can be added to suit different needs.



Versatile functionality

The Swan Curo is a multifaceted toilet and bath chair, ideal for various daily self-care and wellness activities. This allows the child to be as active and involved as possible, promoting quality of life, minimising the need for transfers and the physical workload for the caregiver.

Everyday convenience

Due to a minimalist and small footprint of the frame the toilet and bath chair fits very well into various houses and bathrooms. The simple design makes the chair very user-friendly and lightweight – making it easy for the caregiver to handle and the smooth surfaces easy to clean.



Individual adjustment

Positioning of the user is easily done using the tilt function assisted by a gas spring and the height- and angle-adjustable foot support. To obtain the best possible height for transfer, for caregivers or the toilet, the frame is height-adjustable.



Comfort & support

The shell is carefully designed to enclose the user, which along with the soft PU inlay, ensures high seating comfort. The PU inlay is easy to attach and has high durability. Head support, vests and belts ensure ideal support for the user.

Assisted toileting

Children and youngsters with disabilities unable to maintain a seated position without assistive devices or unable to maintain a standing position while showering, need support for daily toileting.

The Swan Curo is an assistive device for children designed to provide support during toilet and bathing activities. The target group for the device is based on functional ability and not on a specific health condition.



Increased participation

For children and youngsters with disabilities, the environment at home can be a great barrier in participation. The lack of an adaptive toileting environment especially can be a barrier to increased participation. Toilet and bath chairs have shown to have a positive effect on self-care, based on both user activity performance and parent perception.

The Swan Curo is a mobile toilet and bath chair, intended to compensate for the lack of functional abilities, allowing the child to participate in daily toileting activities and be as active and involved as possible. Greater participation in daily living activities, using a toilet and bath chair, can improve self-care and promote dignity and quality of life for children with disabilities.

Access to toileting is a right!

The United Nations highlighted the universal understanding of the importance of access to sanitary facilities and physical access to toileting and handwashing by making access to sanitation a Human Right in 2010. The World Health Organisation states that apart from the obvious healthcare risk, such as diarrhoea, access to sanitation can promote dignity.

R82 supports toilet training!

To support communication about toilet training for children with disabilities R82 have developed communication cards with the Swan Curo, a booklet that introduces children to potty training using the Swan Curo and toileting sheets.

In typically developing children mobility to reach the potty or toilet and independent sitting can be considered essential for initiating toilet training. For children with disabilities assistive devices or physical assistance need to be available to initiate training. A multi-professional approach supporting the child and family in the development of daily routines that might lead to reaching the milestone of day and night-time continence is often necessary.

Toilet training

A way to prepare toddlers for potty training is to talk to them about going to the toilet and using a toilet chair. The R82 Potty book introduces children to potty training using the R82 Swan Curo, and caregivers can read a story about Maria before beginning potty training. Seeing a character, the children can relate to in a book helps them to understand the basics of using a toilet chair. This makes the process more familiar and less daunting, and gets them used to the idea of not wearing nappies.



Communication cards

Children with disabilities can improve their communicative performance by using augmentative and alternative communication such as manual signs, pictures, communication boards, communication books, and talking devices. R82 have developed communication cards with symbols for “Pull down pants”, “Sit on toilet”, “Wipe until clean”, “Pull up pants”, “Flush toilet” and “Wash hands” to help the caregiver to communicate about toileting.

Toileting sheets

The toileting sheets have been developed to support families and healthcare professionals using a structured approach with planned checks, recording, and awarding of daily or weekly successes when using a R82 product for toileting.



Read more about
toilet training



Meet Olivia

Olivia was born in 2015 in China, where she was diagnosed with cerebral palsy. She has been classified using the Gross Motor Function Classification System (GMFCS) level V.

Olivia participates in simple adapted activities using her hands by pushing a button and occasionally holding objects, equivalent to Manual Ability Classification System Level V. Olivia communicates using sounds, gestures, and facial expressions. Morten and Wei use manual signs to reinforce their speech to enhance Olivia's ability to understand them.

Moving to a different part of the world

Olivia and her family moved to Denmark in 2017. Morten and Wei decided to build a new one-family house in Denmark. Taking Olivia's needs for assistive devices and physical assistance into account, they have made a home with an inclusive environment for a child that needs wheeled mobility.

Taking a shower or going to the toilet

For Olivia and her family, daily recurring activities like brushing her teeth, taking a bath, or going to the toilet require the right assistive devices to allow her to be as active and involved as possible.

Olivia's physical impairments restrict voluntary control of movement and her ability to maintain the position of her head and trunk against gravity. The gas spring-supported tilt function changes the seat plan angle from upright to 20° backward tilt. For Olivia, the upright active position supports toileting, while the backward tilted and more relaxed position is great for washing her hair and participating in wellness activities such as having a foot bath.





Read the full case story



1) Olivia on the toilet in an almost upright position with her dad besides her on an Etac Easy shower stool. The Swan Curo, with the narrow footprint fits perfectly into the toilet environment.



2) Olivia sits comfortably in a backward tilted position while Morten gives her a foot massage.

Measurements

When choosing size, it is important to collect the child's measurements and measurements from the toilet and bathroom surroundings. Finally, the measurements are compared to the Swan Curo measurements, to ensure the correct choice of size for the child and the surroundings.



See measurement guide



		1 Without castors	1 With castors	2 With castors	3 With castors	4 With castors
	Effective seat width	220 mm	220 mm	260 mm	320 mm	410 mm
	Effective seat depth	265 mm	265 mm	310 mm	400 mm	450 mm
	Backrest height	380 mm	380 mm	470 mm	525 mm	600 mm
	Max. user mass	35 kg	35 kg	60 kg	85 kg	110 kg
	Seat plane angle (tilt)	0° -20°	0° -20°	0° -20°	0° -20°	0° -19°
	Length	400 mm	495 mm	625 mm	680 mm	680 mm
	Overall width	460 mm	500 mm	530 mm	580 mm	655 mm
	Overall height	670-830 mm	760-920 mm	960-1120 mm	990-1150 mm	1060-1220 mm
	Mass device	7,5 kg	8,5 kg	10,5 kg	11,8 kg	12,8 kg

Further measurements at our website. Size 2, 3 and 4 are always with castors.



molift®

Ergonomic hoists, slings and transfer assistive devices enables safe and comfortable transfers.

Product solution

To suit individual needs, accessories for support can be added to the Swan Curo. To add comfort and functionality, a large range of accessories are available.

By providing the required support to the upper trunk, head and arms, the user is extra comfortable in the chair. See the complete range of accessories at our website.



Gas spring

The gas spring ensures a smooth tilt, decreasing working load for the caregiver when changing position of the chair.

Head support

Add head support according to user needs. This head support is well suited for users who need comprehensive support.

Simple design

The chair is very user-friendly and lightweight – it is easy to operate, easy to clean and easy to handle for the caregiver.

Vests & belts

Attach vests and belts to maintain the trunk position and help keep the pelvis in the back of the seat.

Tray

The tray is ideal for various bathroom activities. It provides upper arm support and helps the child maintain an upright position.

PU inlay

The modern PU inlay ensures an improved seating comfort for the user. The upholstery is easy to attach and detach and easy to clean.

Splash guards

Two options for splash guards are available as accessories for the R82 Swan Curo.

Lockable castors

For safety reasons during transfer etc., the chair is equipped with lockable castors. Larger castors are available as accessories.

Height-adjustable

The frame is height-adjustable to accommodate a wide range of toilets.

Rod for foot support

The angle adjustable foot rod makes it easy to adjust foot support.



See instruction videos

Accessories



PU inlay, blue, complete
8706x-61



PU inlay, blue, for back
8706x-61



PU inlay, blue, for seat
870xx-61



Seat infill, blue
8707x-61



Footplate
87145-x



Rod for foot support
8702x



Commode pan
87080x



Potty ring
87080x



Castors, lockable, set of 4
75 mm - 87100-20, 100 mm - 87100-S,
125 mm - 880100-125



Tray, ABS
870xx



Swing-away armrest / tray
87013x-61



Push handle
one size - 880110



Splash guard, 13 cm, blue
one size - 880508-0561



Splash guard, low, blue
one size - 880508-0861



B3240 2.0 | Brochure, R82 Swan Curo - Global EN | 2022 | Reserving our rights for printer's error or discontinued products.
Please find an updated edition on our website: R82 A/S All rights reserved.

Founded in 1973, Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our ambition is to provide solutions that optimise quality of life for the individual, family members and caregivers.

For the latest news and continuously updated product information – visit: www.etac.com

R82 A/S

 +45 79 68 58 88

 R82@etac.com

 www.etac.com


Creating Possibilities